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AW-00432 Rev. 0 04/2021

EN INSTRUCTION FOR USE

Lubricating eye drops with Sodium Hyaluronate 0.2% solution

Composition

Sodium hyaluronate 0.2%, glycerin, sodium chloride, boric acid, sodium tetraborate, calcium chloride, potassium chloride, magnesium chloride, highly purified water.

Description

Xailin® Plus is a lubricating eye drops with 0.2% sodium hyaluronate solution, obtained by biotechnological synthesis. Sodium hyaluronate, due to its specific physical, viscoelastic properties and the ability to adhere to the cornea, improves tear film stability and restores comfort of vision.

Indications

Moisturises and alleviates symptoms associated with dry eye syndrome and irritations, redness and eye fatigue caused by prolonged screen use, exposure to environmental factors such as wind, sun, dry air, salty water, smoke, dust, air conditioning and heating. Prevents symptoms of eye irritation caused by contact lens use.

Who are Xailin® Plus eye drops intended for?

Xailin® Plus is indicated for children over one month of age and adults.

How often should Xailin® Plus eye drops be used?

Xailin® Plus eye drops can be used daily, including with all types of contact lenses, as the drops do not contain any preservatives. It is recommended to apply one drop into the conjunctival sac of the eye 3-4 times a day. You can apply more often if needed.

How to use Xailin® Plus eye drops?

Xailin® Plus eye drops are for external use only, applied to the conjunctival sac of the eye.

1. Prior to administering the preparation wash hands thoroughly.
2. Remove the cap and apply into the conjunctival sac of the eye. Do not touch the surface of the eye with dropper tip.
3. Tilt your head back slightly.
4. Hold the bottle face down (vertically).
5. Pull your lower eyelid down slightly, pressing the bottle, squeeze one drop into the conjunctival sac.
6. Close your eyelids and spread the drop across the eye surface by blinking.
7. Repeat the same activities for the other eye.
8. Put the protective cap on the dropper after use. During application, be careful not to touch the surface of the eye and eyelid with the tip of the dropper.
9. After applying the drops, do not rinse the surface of the eye with water or any other liquid.

When Xailin® Plus eye drops are contraindicated?

- Do not use in the case of hypersensitivity to any of its components.
- If an allergic reaction occurs, discontinue use.
- Do not apply in the case of bacterial, fungal or viral inflammation of the eye.

How to store Xailin® Plus eye drops?

- Keep out of sight and reach of children.
- Keep away from sources of heat.
- Store in a dry place at a temperature of 5°C to 35°C.
- Do not use the product after the expiration date printed on the package.
- From the date of first opening, use no longer than 30 days continuously and no longer than 90 days discontinuously (taking into account several days breaks during application).
- Discard if any unused solution.

Warnings and precautions:

- The preparation is intended for external use only.
- Do not use if package is damaged.
- Never touch your eye or any other surface with the tip of the dropper, this may contaminate the solution.
- In the case of using other ophthalmic solutions wait at least 15 minutes before the application of Xailin® Tears eye drops.
- The preparation should be administered last.
- The reusable medical device should be used by one patient.

Reporting medical incidents or adverse reactions:

If you get any side effects, please tell your doctor, pharmacist or nurse.

Package: Bottle of 10ml in carton unit box. Medical device.

Rev. 0 04/2021

SUPROBION Sp. z o.o., Sp. k.
ul. Kielczowska 2, 55-095 Mirków - Poland

Distributed by VISUfarma B.V.
Amstelplein 1, 1096 HA, Amsterdam - Netherlands
info@visufarma.com

STERILE A

CE 0051

AW-00432 Rev. 0 04/2021