PRODUCT NAME: IFU NAVIBLEF DAILY CARE 50ml FR DE EN CHN		LANGUAGES: FR DE EN CHN		SIZE: 230x170 mm	
MATERIAL CODE: AW-00723		REVISION LEVEL: 1		ARTWORKER: Barbara Nuvoile	
COLOUR #1:  Black	COLOUR #4: 	COLOUR #7:  --	PROGRAMME: Adobe Illustrator		
COLOUR #2: 	COLOUR #5: 	COLOUR #8:  --			
COLOUR #3: 	COLOUR #6: 	COLOUR #9: 	FONTS: Microsoft YaHei UI, Myriad Pro, Futura		
FINAL APPROVALS					
COUNTRY MANAGER #1:		COUNTRY MANAGER #2:		REGULATORY:	
COUNTRY MANAGER #3:		COUNTRY MANAGER #4:			
NOTES:					

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EN EYELID CLEANSING FOAM
FOR OCULAR SECRETIONS

For External Use Only

COMPOSITION: Melaleuca Alternifolia (Tea Tree) Essential Oil, Anthemis Nobilis (Chamomile) Essential Oil, D-Panthenol, Allantoin, Taurine, Cocamidopropyl betaine, Polysorbate 80, Boric Acid, Disodium EDTA, Tris (hydroxymethyl) aminomethane, Sodium N-hydroxymethylglycinate and purified Water.

INDICATIONS: If you have dry, scratchy or irritated eyes, the problem may not be your eyes, but rather your eyelids. **NAVIBLEF® TTO DAILY CARE** is designed to treat irritated eyelids affected by pathologies such as dryness, allergies, blepharitis and blepharokonjunktivitis due to the presence of Demodex (eyelid mites) and/or bacterial infestations.

The effect of **NAVIBLEF® TTO DAILY CARE** is obtained through the mechanical removal of excessive oil, pollen, and other debris from the eyelids. In case of persisting secretions and residues, daily use is recommended. This foam can also be used to clean the base of the eyelashes and eyelid margins before and after surgery.

NAVIBLE® TTO DAILY CARE is recommended to be used as maintenance care, for patients who have followed the blepharitis treatment with NAVIBLE® TTO INTENSIVE CARE.

DESCRIPTION: NAVIBLE® TTO DAILY CARE contains Tea Tree Oil, which, thanks to its oily texture, helps to eliminate secretions of eyelids and eyelashes.

NAVIBLE® TTO DAILY CARE contains Allantoin to help keep the skin hydrated.

WHAT IS BLEPHARITIS?: It is a chronic inflammation of the eyelids and is one of the most common disorders of the eye, usually due to the presence of Demodex and bacterial infestations. This is often the main cause of eye discomfort, redness and tearing. Other eye symptoms of blepharitis include burning, itching, light sensitivity, irritation, sandy and gritty sensation that is more accentuated upon awakening.

Blepharitis is a chronic disease for which there is no cure and requires a long-term treatment, hygiene and maintenance to keep it under control and prevent subsequent related complications linked to this type of conditions.

WHAT IS DRY EYE?: It is known as a disorder of the tear film due to tear deficiency or excessive tear evaporation, which can cause damage to the interpalpebral and ocular surface.

IN CASE OF EYE SURGERY: Before or after eye surgery, there is a risk of infection due to the presence of bacteria on the eyelids and eyelashes, so it is important to follow the recommendations of your eye specialist with regard to the hygiene and care to follow.

PACKAGING:

Bottle of 50ml with a pump dispenser [REF] F50NTTD475

Bottle of 5ml with a pump dispenser [REF] F5NTTD476

PROPERTIES: NAVIBLE® TTO DAILY CARE is a non-greasy and non-irritant formulation, for topical use.

INSTRUCTIONS FOR USE:

1- Before the application of the foam, wash your hands and then clean the fingertips with NAVIBLE® TTO DAILY CARE.

2- Shake the bottle before application to produce the foam. Press the delivery pump and place the foam on the clean fingertips. When used before or after surgery, or in case of sensitive eyes, it is recommended to place the foam on sterile gauze.

3- Close the eye and massage the eyelid and eyelashes with NAVIBLE® TTO DAILY CARE. Do not touch or open the eyes during the application. Leave the foam for approximately 60 to 80 seconds.

4- After each application rinse the eyelashes and eyelids with warm water.

Note: Repeat the same steps for the other eye. Repeat this treatment 2 - 3 times a day for an efficient maintenance care of eyelids and eyelashes.

PRECAUTIONS: • Always wash your hands before and after use; • Remove contact lenses before the application of the foam. Ensure all foam and residue are removed before re-inserting lenses; • Keep your eyes closed during the application; • The product is intended for the application on eyelids and eyelashes. Do not apply directly in the eyes; • For external use only; • Keep out of the sight and reach of children; • Don't use the product in children under 6 years old. For children over 6 years old the application must be performed by an

adult and only after your specialist's advice; • In case of use in pregnant or lactating women it is advised to consult your specialist beforehand; • Use hypoallergenic eye makeup and delicate skin lotions around your eyes during the use; • Do not use in case of known hypersensitivity to any of the constituents; • If a reaction to the product occurs, stop use immediately, rinse your eyes with water and contact your eye specialist if the symptom continues; • Discard any remaining solution 6 months after first opening; • Do not use if the bottle is damaged; • Do not use after the expiry date.

STORAGE: Store between 5°C and 35°C and protect from a direct source of light.

PRESENTATIONS: NAVIBLE® TTO is presented in two formulations: • NAVIBLE® TTO INTENSIVE CARE to eliminate severe ocular secretions and debris due to infections caused by bacteria and Demodex (red box); • NAVIBLE® TTO DAILY CARE for the maintenance care of eyelids and eyelashes (blue box).

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VISUfarma B.V.
International Associates Limited