PRODUCT NAME: IFU NAVIBLEF INTENSIVE CARE 50 ml FR DE EN CHN		LANGUAGES: FR DE EN CHN		SIZE: 230x170 mm	
MATERIAL CODE: AW-00723		REVISION LEVEL: 1		ARTWORKER: Barbara Nuvole	
COLOUR #1:  Black		COLOUR #4: 		COLOUR #7:  --	
COLOUR #2: 		COLOUR #5: 		COLOUR #8:  --	
COLOUR #3: 		COLOUR #6: 		COLOUR #9: 	
PROGRAMME: Adobe Illustrator					
FONTS: Microsoft YaHei UI, Myriad Pro, Futura					
FINAL APPROVALS					
COUNTRY MANAGER #1:		COUNTRY MANAGER #2:			REGULATORY:
COUNTRY MANAGER #3:		COUNTRY MANAGER #4:			
NOTES:					

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EN BLEPHARITIS EYELID FOAM HELPS TO ELIMINATE SECRETIONS AND DEBRIS FROM EYELIDS AND EYELASHES COLONISED BY BACTERIA AND DEMODEX

For External Use Only

COMPOSITION: Melaleuca Alternifolia (Tea Tree) Essential Oil, Anthemis Nobilis (Chamomile) Essential Oil, D-Panthenol, Aloe Vera, Taurine, Chamomile aroma, Tris(Hydroxymethyl) aminomethane, Boric Acid, Disodium EDTA, Sodium N-hydroxymethylglycinate, Cocamidopropyl betaine, PEG-40 Hydrogenated Castor Oil, PPG-26 Buteth 26, trideceth-9 and purified Water.

INDICATIONS: NAVIBLEF® TTO INTENSIVE CARE is indicated in the treatment of irritated eyelids and eyelashes through the mechanical removal of severe ocular secretions. These are caused by inflammatory conditions such as dryness, contact lens wearing, allergies, blepharitis and blepharokonjunctivitis resulting from the presence of Demodex (eyelid mites) or bacterial infestations.

This foam can also be used to clean the base of the eyelashes and eyelid margins before and after surgery.

DESCRIPTION: NAVIBLER® TTO INTENSIVE CARE contains a high concentration of Tea Tree Oil, which, thanks to its oily texture, helps to eliminate the most stubborn secretions of eyelids and eyelashes. NAVIBLER® TTO INTENSIVE CARE contains Aloe Vera to help keep the skin hydrated.

WHAT IS BLEPHARITIS?: It is a chronic inflammation of the eyelids and is one of the most common disorders of the eye, usually due to the presence of Demodex and bacterial infestations. This is often the main cause of eye discomfort, redness and tearing. Other eye symptoms of blepharitis include burning, itching, light sensitivity and irritation, sandy and gritty sensation that is more accentuated upon awakening.

Blepharitis is a chronic disease for which there is no cure and requires a long-term treatment, hygiene and maintenance to keep it under control and prevent subsequent related complications linked to this type of conditions.

PACKAGING:

Bottle of 50ml with a pump dispenser [REF] F50NTT1477

Bottle of 5ml with a pump dispenser [REF] F5NTT1478

PROPERTIES: NAVIBLER® TTO INTENSIVE CARE is a non-greasy and non-irritant formulation, for topical use.

INSTRUCTIONS FOR USE:

1- Before the application of the foam, wash your hands and then clean the fingertips with NAVIBLER® TTO INTENSIVE CARE.

2- Shake the bottle before application to produce the foam. Press the delivery pump and place the foam on the clean fingertips. When used before or after surgery, or in case of sensitive eyes, it is recommended to place the foam on sterile gauze.

3- Close the eye and massage the eyelid and eyelashes with NAVIBLER® TTO INTENSIVE CARE. Do not touch or open the eyes during the application. Let the foam dry before opening your eye to avoid the product entering into the eye. Leave the product on for at least 3 to 5 minutes.

4- After each application rinse the eyelashes and eyelids with warm water.

Note: Repeat the same steps for the other eye. Repeat this treatment 2 - 3 times a day for a period between 7 to 14 days, for maximum efficacy

PRECAUTIONS:

- Always wash your hands before and after use;
- Remove contact lenses before the application of the foam. Ensure all foam and residue are removed before re-inserting lenses;
- **Keep your eyes closed and let the foam dry before opening the eyes;** • The product is intended for the application on eyelids and eyelashes. Do not apply directly in the eyes;
- **For external use only;** • Keep out of the sight and reach of children;
- Don't use the product in children under 6 years old. For children over 6 years old the application must be performed by an adult and only after your specialist's advice;
- In case of use in pregnant or lactating women it is advised to consult your specialist beforehand;
- Use hypoallergenic eye makeup and delicate skin lotions around your eyes during the use;
- Do not use in case of known hypersensitivity to any of the constituents of the product;
- If a reaction to the product occurs, stop use immediately, rinse your eyes with water and contact your eye specialist if the symptom continues;
- Discard remaining solution within 3 months after first opening;

• Do not use if the bottle is damaged; • Do not use after the expiry date.

STORAGE: Store between 5°C and 35°C and protect from a direct source of light.

PRESENTATIONS: NAVIBLER® TTO is presented in two formulations:

- NAVIBLER® TTO INTENSIVE CARE to eliminate severe ocular secretions and debris due to infections caused by bacteria and Demodex (red box);

- NAVIBLER® TTO DAILY CARE for the maintenance care of eyelids and eyelashes (blue box).

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VISUfarma B.V.
International Associates Limited