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V1 - Rev. 11-01-2018

EN **BLEPHARITIS EYELID FOAM.**
HELPS TO ELIMINATE SECRETIONS AND DEBRIS FROM EYELIDS AND EYELASHES
COLONIZED BY BACTERIA AND DEMODEX.

For external use only

COMPOSITION

D-Pantenol, Aloe Vera, Taurine, Melaleuca Alternifolia (Tea Tree Oil), Anthemis Nobilis (Camomile E.O.), Aroma Camomile and purified Water.

DESCRIPTION AND INDICATIONS

NAVIBLEF® INTENSIVE CARE is indicated for hygiene and mechanical removal of severe ocular secretions and debris from eyelashes and eyelids. These are caused by inflammatory conditions due to blepharitis and blepharoconjunctivitis, presence of bacteria or Demodex (eyelid mites), dry eyes, contact lenses wear or ocular surgery.

NAVIBLEF® INTENSIVE CARE can help reducing the risk of infections caused by bacteria and Demodex present on eyelids and eyelashes, also before and after ocular surgery.

PACKAGING

Bottle of 50 ml with a pump dispenser  F50N1186

Bottle of 5 ml with a pump dispenser  F5N1185

PROPERTIES

NAVIBLEF® INTENSIVE CARE is a non-greasy and non-irritant formulation, for topical use.

WHAT IS BLEPHARITIS?

It is a chronic inflammation of the eyelids and is one of the most common disorders of the eye. This is often the main cause of eye discomfort, redness and tearing. Other eye symptoms of blepharitis include burning, itching, light sensitivity and irritation, sandy and gritty sensation that is more accentuated upon awakening.

Blepharitis is a chronic disease for which there is no cure and requires a long-term treatment, hygiene and maintenance to keep it under control and prevent subsequent related complications linked to this type of conditions.

DIRECTIONS FOR USE

1 - Before the application of the foam, wash your hands and then clean the fingertips with NAVIBLEF® INTENSIVE CARE.

- 2 - Press the delivery pump and place the foam on the clean fingertips. When used before or after surgery it is recommended to place the foam on sterile gauze.
- 3 - Close the eye and massage the eyelid and eyelashes with NAVIBLEF® INTENSIVE CARE. Do not touch or open the eyes during the application. Let the foam dry before opening your eye to avoid the product entering into the eye. Leave the product on for at least 3 to 5 minutes.
- 4 - After each application rinse the eyelashes and eyelids with warm water.

Note: Repeat the same steps for the other eye.

Repeat this treatment 2 - 3 times a day for a period between 7 to 14 days, for maximum efficacy.

PRECAUTIONS

- Always wash your hands before and after use.
- Remove contact lenses before the application of the foam. Ensure all foam residue is removed before re-inserting lenses.
- Keep your eyes closed during the application.
- The product is intended for the application on eyelids and eyelashes. **Do not apply directly in the eyes.**
- **For external use only.**
- Keep out of the sight and reach of children.
- Do not use for children under 3 years.
- Use hypoallergenic eye makeup and delicate skin lotions around your eyes during the use.
- Do not use in case of known hypersensitivity to one of the constituents of the product.
- If irritation occurs, stop use immediately, rinse your eyes with water and contact your eye specialist if the irritation continues.
- Discard remaining solution within 6 months after first opening.
- **Do not use after the expiry date.**

STORAGE

- Store the product in a dry place between 5 and 35°C.

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EN


PRESENTATIONS: NAVIBLEF® is presented in two formulations:

- NAVIBLEF® INTENSIVE CARE to eliminate severe ocular secretions and debris due to infections caused by bacteria and Demodex (Red box).
- NAVIBLEF® DAILY CARE for long term daily hygiene and prevention of ocular secretions and debris due to infections caused by bacteria and Demodex (Blue box).

Distributor in the UK:
VISUfarma International,
2405, route Des Dolines - Drakkar D,
06560 Valbonne, FRANCE

VISUfarma

NOVAX® PHARMA
Le Coronado - 20, Av. de Fontvieille
MC 98000 MONACO
www.novaxpharma.com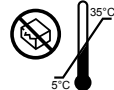

V1 - Rev. 11-01-2018