EN

Description

Xailin® HA is a preserved sterile hypotonic multi dose lubricating eye drop, containing sodium hyaluronate at 0.2%.

Xailin® HA maintains and restores the physiological conditions of the eye surface, protects its cells, induces a long lasting comfort.

Indications

Treatment of dry eye symptoms.

Intended purpose

Xailin® HA brings prompt and lasting relief in case of sensation of dryness, irritated and tired eyes caused by dry eye disease and by external factors such as prolonged exposure to video screens, air conditioning, dust, pollution, smoke, etc.

Xailin® HA can be used by wearers of all types of corrective contact lenses.

Composition

Sodium Hyaluronate 0.2 %, Sodium chloride, Boric Acid, Borax, Dequest, Sodium Perborate (0.03%) and Purified Water.

Packaging

10 ml into a multidose ophthalmic container.

Instruction for use

- Wash and rinse your hands thoroughly.
- Remove the protective cap. Make sure that the tip of the bottle does not touch your fingers.
- Lean your head back slightly and pull your lower eyelid down slightly with a finger of your free hand and hold the bottle vertically above your eye with the nozzle pointing downwards. Then gently squeeze the bottle to apply one or two drops into each eye, or according to the specialist's advice. Avoid contact of the top of the bottle with eye or your fingers to prevent any possible contamination of the solution.
- Replace the protective cap after use.
- The product can be used whenever necessary.

Precautions

- Do not use if the bottle is damaged.
- Do not share the bottle to prevent cross contamination.

Warnings

- If you have to apply additional treatments to the eye, an application interval of at least 15 minutes should be left between the two medications.
- To prevent possible contamination, do not let the tip of the dropper come into contact with any surface.
- Do not use if you are hypersensitive (allergic) to any of the constituents.
- Rare cases of mild eye irritation are possible after product instillation. Stop using it if you experience pain, redness or any change in vision following the instillation. Ask a doctor for advice if the symptoms persist or get worse.
- If you experience blurred vision after application, wait until your vision is clear before driving or using tools or machinery.
- Do not swallow the solution.
- Keep out of reach of children.
- If you experience a persistent problem, please consult your doctor/optician.
- Do not instill **Xailin® HA** in the eye(s) together with other substances as their effect may be altered.

Storage Conditions

- Store the bottle at a temperature below 30°C.
- Do not use the solution after the expiry date.
- After the bottle has been opened, the solution must be used within 90 days. Any residual product must be disposed of.

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Tubilux Pharma S.p.A
VISufarma B.V.