

VisuEVO®

For immediate, long-lasting relief from dry eye

Sterile eye drops without preservatives

INSTRUCTION FOR USE

Please read this information carefully because it will help you to use VisuEVO® properly.

What is VisuEVO®?

VisuEVO® is an innovative formulation containing omega-3 essential fatty acids, Vitamins A and D and ultra-filtered phospholipids. VisuEVO® reduces excessive evaporation of the tear film, which is caused by inflammatory conditions on the surface of the eye, due to the presence of liposomes (phospholipids), Omega-3 and Vitamin D.

Why do we need VisuEVO®?

VisuEVO® is indicated:

- for all forms of evaporative dry eye, to improve the stability of the lipid layer by reducing the evaporation of the aqueous phase, i.e. the tear film (which can be caused, for example, by prolonged use of computers, reading, driving, the use of medicines, ocular allergies and contact lenses);
- as a pre-treatment for eye surgery, in order to help with the restoration of the lipid integrity on the surface of the eye;
- for forms of dry eye caused by a reduced secretion of lipids from the Meibomian glands;
- for cases of dry eye caused by a mucin deficiency;
- for cases in which dry eye has worsened as a result of the use of preserved eye drops.

Composition:

Liposomes from soya phospholipids, algae oil (DHA+EPA), Vitamin A palmitate, Vitamin D, disodium EDTA, PEG 400, Boric acid, Sodium tetraborate, buffered isotonic buffered with pH 7.20.

When should you apply VisuEVO®?

Vitamin A and phospholipids are natural constituents of the tear film. Their antioxidant action prevents damage caused by reactive oxygen species and facilitates homoeostasis and the retention of the tear film, as well as the control of its evaporation.


Omega-3, Vitamin D and Phospholipids improve the osmolarity of the tear film, protecting it from inflammatory conditions caused by excessive evaporation.

How to apply VisuEVO® ?



STEP 1

Hold the bottle just under the cap and turn the cap to open it.

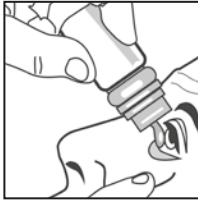

STEP 2

Instill 1 drop into the conjunctival sac of each eye by gently pulling down the lower eyelid and looking upwards. Dispense the product by gently squeezing the bottle. Do not touch the eye with the tip of the bottle.



STEP 3

Close the bottle immediately after use. The solution can continue to be used within 60 days after the first opening.

Only one person should be treated with one particular bottle of VisuEVO®.

What else do you need to know?

In case of concomitant therapies, instill VisuEVO® at least 10 minutes after the administration of other medications.

VisuEVO® can be instilled while wearing contact lenses;

Wash your hands before applying the product.


How often can you use VisuEVO®?

Instill 1 drop of VisuEVO® into the conjunctival sac of each eye 3 times a day or according to the directions of your ophthalmologist.

When should you not use VisuEVO®?

Do not use the product if you are allergic or hypersensitive to any of its components;

If you have any adverse reactions, discontinue its use and consult your ophthalmologist;

Do not use beyond the expiry date ();

Do not use the bottle if it is damaged.

What you need to know about storing VisuEVO®?

Store the product at room temperature (15-25°C);

Keep the product away from light, heat and out of the reach of children;

The product remains sterile for 60 days after first opening of the bottle;

The expiry date refers to the product in an intact, correctly stored product.

The following package sizes are available:

3 ml of preservative free eye drops in Novelia® System.

5 ml of preservative free eye drops in Novelia® System.

10 ml of preservative free eye drops in Novelia® System.

Manufacturer

VISUfarma S.p.A.

Via Alberto Cadlolo, 21 00136 Roma, Italy

info@visufarma.it

Local Responsible Person:

Kerry Pharma (Hong Kong) Limited

16/F Kerry Cargo Centre, 55 Wing Kei Road, Kwai Chung, N.T.

Tel No.: 2211 3799

Last revision of the text: Jan 2021

