



**Night-time relief  
of dry eye sensations  
Preservative Free**  
5 g Sales pack - 3.5 g Sample pack

### EN Instruction for use

It is important to read this leaflet carefully before using **Xailin® Night**. The information provided summarises the important points about the product but if you have any further questions then please ask your doctor, pharmacist or optician.

#### 1. What is Xailin® Night?

**Xailin® Night** is a multi-dose preservative free, white, translucent, free from foreign matters, soft ointment for night-time relief of dry eye sensations. The pack contains 5 g (Sales) or 3.5 g (Sample) of sterile eye ointment.

#### 2. Composition:

**Xailin® Night** contains white soft paraffin 57.3% w/w, white mineral oil 42.5% w/w, and lanolin alcohols 0.2% w/w which produce a transparent, lubricating and moisture-retaining film on the surface of the eye.

#### 3. What is Xailin® Night used for?

**Xailin® Night** provides strong soothing night-time relief of dry eye sensations, as it acts as a barrier against moisture loss. Ideal for use at bedtime.

**Xailin® Night** provides soothing relief from the symptoms of dry eyes, such as soreness, irritation or a gritty sensation. Dry eye symptoms occur as a result of ocular surface disorders that can be caused by a number of different reasons where the normal tears are either of poor quality or produced in too little quantity to keep the eye sufficiently moist.

There is no age restriction for using **Xailin® Night**.

**Xailin® Night** may be used during pregnancy and breast-feeding.

#### 4. How to use:

Before opening a new tube of **Xailin® Night**, please check that the safety strip on the cap of the tube is unbroken.

- Wash and dry your hands well before use.
- Tear off safety strip to break the seal.
- Remove the outer cap.
- If you are a contact lens wearer remove your lenses.
- Tilt your head back and look up.
- Gently pull down the lower lid of the eye to form a small pocket between your eyelid and your eye.
- Hold the tube between the thumb and middle finger of the other hand, turn the tube upside down near to the eye, and try not to touch the eye with the nozzle.
- Apply enough pressure to the tube to release a thin line of ointment about half an inch (1 centimetre), or as directed by your doctor, pharmacist or optician.
- Blink your eye a few times.
- If you missed the eye, then insert another line of ointment.
- Carefully wipe away any excess **Xailin® Night** around the eyelid with a tissue.
- Repeat the process for the other eye, if required.
- Replace the outer cap on the container, do not touch the applicator tip with anything, including your eye or your fingers.
- Leave at least 30 minutes between administrations of any other ophthalmic drugs.

If you are unsure about how to use **Xailin® Night** or if your eye conditions persist, consult your doctor, pharmacist or optician.

#### 5. How much to use?

Apply a thin line of ointment about half an inch (1 centimeter), placed in each eye whenever needed.

#### 6. Precautions and warnings:

- Do not use if the safety strip on the cap of the tube is broken.
- Do not use if you are allergic to lanolin alcohols or any of the ingredients.
- Do not use with contact lenses.
- Do not share **Xailin® Night** with any other person.
- **Xailin® Night** can cause transient mild burning sensation and or blurring of vision after application. Wait until your vision is clear before driving or using machinery.
- Do not use after the expiry date shown on the carton or printed on the crimp of the tube.
- Store between 1°C and 25°C.
- Discard the contents 60 days after first opening. Keep out of the reach and sight of children.
- External ocular use only, do not swallow.
- Dispose of the used tube according to your local waste management regulations.

#### 7. Contraindications

Do not use if you are allergic to any of the ingredients listed in section 2-Composition.

#### 8. Possible Side Effects

- Transient blurring of vision
- Mild discomfort in eye
- Possible allergic reaction

If you experience any additional adverse events, consult your healthcare professional and report to Medicom Healthcare Ltd.

#### 9. How to store Xailin® Night?

- Do not store above 25°C.
- Discard the **Xailin® Night** 60 days after first opening.
- Dispose of the tube according to your local waste management regulations.

This leaflet applies to **Xailin® Night** and not to any other product.

Last revision of the text: May 2023

**Xailin®** is a trademark of VISUfarma

	Consult instructions for use
	Sterilised using aseptic processing techniques
	Do not store above 25°C. Do not freeze the product.
	Discard the content 60 days after first opening
	Do not use if package is damaged and consult instructions for use
	Medical device symbol

#### Distributed by VISUfarma BV

Amstelplein 1, 1096 HA,  
Amsterdam, Netherlands  
info@visufarma.com

**Medicom Healthcare Ltd,**  
Lynton House, 7-12 Tavistock  
Square, Kings Cross,  
London, WC1H 9LT, U.K.

**UK  
CA0120**

13-LE05.0UKCA-10-GB-PVI  
AW-00627 Rev. 1



PRODUCT NAME: INST XAILIN NIGHT EN-UKCA	LANGUAGES: EN	SIZE: 148x420 mm
MATERIAL CODE: AW-00627	REVISION LEVEL: 1	ARTWORKER: Barbara Nuvole
COLOUR #1:  black	COLOUR #4: <input type="checkbox"/>	PROGRAMME: ADOBE CC
COLOUR #2: <input type="checkbox"/>	COLOUR #5: <input type="checkbox"/>	FONTS: Myriad Pro Semicondensed
COLOUR #3: <input type="checkbox"/>	COLOUR #6: <input type="checkbox"/>	

#### FINAL APPROVALS

COUNTRY MANAGER #1:	COUNTRY MANAGER #2:	REGULATORY:
COUNTRY MANAGER #3:	COUNTRY MANAGER #4:	

NOTES: