Eye drops for the symptomatic relief of allergy-related irritation and dryness

Sterile eye drops without preservative

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EN INSTRUCTION FOR USE

What are Xailin® Ectoine eye drops?
The innovative combination of four active substances contained in the eye drops: ectoine, sodium hyaluronate, fucoidan and Perilla frutescens extract, effectively soothes inflammation caused by allergies.

What are Xailin® Ectoine eye drops used for?
Xailin® Ectoine eye drops prevent and support the treatment of symptoms of seasonal and chronic allergic conjunctivitis. They soothe irritations, have a moisturising effect and support the regeneration of the protective barrier of the eye surface.

When should Xailin® Ectoine eye drops be used?
Xailin® Ectoine should be used in case of eye irritation caused by allergens, i.e. pollen, dust mites, animal hair or mould. Thanks to the unique combination of four active substances, the eye drops both intensely moisturise and quickly soothe the bothersome symptoms of allergic conjunctivitis (i.e. redness, itching, burning or watery eyes).

How do the active ingredients of Xailin® Ectoine eye drops work?

- Sodium hyaluronate** - a naturally occurring polysaccharide in the human body, including the vitreous body of the eye, is commonly known as a highly moisturising substance. The mucoadhesive properties of hyaluronic acid ensure better adherence of the preparation to the eye surface, which contributes to faster healing of the corneal epithelium, and thanks to the viscoelastic properties, it increases the stability of the tear film, which is washed out much slower during blinking.
- Ectoine** - a natural substance produced in the fermentation process by microorganisms living in extreme environmental conditions - extremophiles. Ectoine belongs to substances with hygroscopic (water binding / moisture absorption) and osmoprotective properties, thanks to which it moisturises the surface of the nasal mucosa, protects epithelial cells and stabilises cell membranes. Stabilisation of cell membranes, and consequently enhancement of tissue barrier function, may reduce allergen-membrane interactions and alleviate symptoms caused by inflammation.
- Fucoidan** - is a sulfonated polysaccharide of natural origin, which is part of the cell membranes of brown algae of the fucus type and aquatic animals (e.g. sea cucumbers and sea urchins). It is thanks to Fucoidan that the seaweed does not dry out during low tide and does not die under the influence of the sun. The mucus with which they are covered also provides comprehensive protection against the harmful effects of external factors. Fucoidan is known to be used as a moisturising, hydrogel forming and humectant component.
- Perilla frutescens** - is an annual oil plant from the Lamiaceae family. It exhibits a number of properties – from moisturising to soothing the symptoms associated with the occurrence of an allergic reaction in the mucous membranes.

Who are Xailin® Ectoine eye drops intended for?
Xailin® Ectoine eye drops are intended for children over 6 years of age and adults.

How much Xailin® Ectoine eye drops should you use?
It is recommended to instill 1 drop 3 times a day into the conjunctival sac of each eye. With severe ailments, you can apply more often.

How to use Xailin® Ectoine eye drops?
Xailin® Ectoine eye drops should be applied into the conjunctival sac.

When Xailin® Ectoine eye drops are contraindicated?

- Do not use in the case of hypersensitivity to any of its components.
- If an allergic reaction occurs, discontinue use.

Composition: sodium hyaluronate, ectoine, fucoidan, perilla extract (Perilla frutescens), boric acid, sodium tetraborate, sodium chloride, highly purified water.

How to store Xailin® Ectoine eye drops?

- Keep out of sight and reach of children.
- Keep away from sources of heat.
- Store in a dry place at a temperature of 5°C to 35°C.
- Do not use the product after the expiration date printed on the package.
- The shelf life of the product is 3 months after first opening the container.
- The bottle provides 1 month's treatment following the recommended dosing. Discard the bottle 3 months after first opening.
- Discard any unused solution.

Warnings and precautions:

- The preparation is intended for external use only.
- Do not use if package is damaged.
- Never touch your eye or any other surface with the tip of the dropper, this may contaminate the solution.
- In the case of using other ophthalmic solutions you should wait at least 15 minutes before the application of Xailin® Ectoine eye drops.
- Xailin® Ectoine should be administered last.
- The reusable medical device should be used by one patient.

Adverse effects: If adverse effects occur during the use of a medical device, they are usually mild and of a short duration. These can include discomfort, irritation, burning, stinging, itching and blurred vision.

Reporting medical incidents or adverse reactions: If you get any side effects, please tell your doctor, pharmacist or nurse. You can also report adverse effects on the website:
<https://www.visufarma.com/contact/> or to the address: VISUfarma B.V. Amstelplein 1, 1096 HA, Amsterdam.

Package: Bottle of 10ml in carton unit box.
Date of instruction preparation: September 2024

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VISUfarma

8 - 9 - 10 - 11

EN 1) Prior to administering the preparation wash hands thoroughly. 2) Remove the cap and apply into the conjunctival sac of the eye. Do not touch the surface of the eye with dropper tip. 3) Tilt your head back slightly. 4) Hold the bottle face down (vertically). 5) Pull your lower eyelid down slightly, pressing the bottle, squeeze one drop into the conjunctival sac. 6) Close your eyelids and spread a drop across the eye surface by blinking. 7) Repeat the same activities for the other eye. 8) Put the protective cap on the dropper after use. During applying, be careful not to touch the surface of the eye and eyelid with the tip of the dropper. 9) After applying the drops, do not rinse the surface of the eye with water or any other liquid. 10) Remember that a few seconds may pass between the squeezing of the bottle and the flow of the drop in. Do not apply too much pressure. 11) Before closing, point the bottle with the applicator downwards and shake vigorously, without touching the tip of the dropper, in order to remove the remaining drops. Wait a moment before putting the cap on for the bottle to vent.

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VISUFarma B.V. - Amstelplein 1, 1096 HA, Amsterdam