Front side

Xailin®
Night

Lubricating Eye Ointment • Preservative Free
Unguento Lubrificante Oftalmico • Senza Conservanti
Kayganlaştırıcı Göz Merhemi • Koruyucu İçermeyen
Unguent Oftalmic Lubrifiant • Fără Conservanți

EN INSTRUCTIONS FOR USE

It is important to read this leaflet carefully before using Xailin® Night Lubricating Eye Ointment. The information provided, summarizes the important points about the product but if you have any further questions then please ask your doctor, pharmacist or optician.

What is Xailin® Night? Xailin® Night Lubricating Eye Ointment is an off-white to pale yellow, multi-dose, preservative free smooth ointment. **Intended Purpose:** Xailin® Night is a soft ointment that spreads easily and provides lubrication to the eye surface. It improves the tear film and hence relieves the sensations of burning, soreness, irritation and dry eyes.

Composition: White Soft Paraffin 57.3% w/w; Liquid Paraffin 42.5% w/w; Wool Alcohols 0.2% w/w.

When should it be used? Xailin® Night Lubricating Eye Ointment is used to lubricate and protect your eyes in certain eye conditions, or during or after surgery.

How to use: • Wash hands before administering the product. • Gently pull the lower eye lid of the affected eye and apply a thin line of the ointment to the inside of the eye. • Repeat the process for the other eye, if required. • Repeat this treatment as often as needed or as directed by your doctor. • To avoid contamination, do not touch the tip to any surface, including the eye. • To avoid injury to the eye, do not allow the tube tip to come into contact with the eye during the application of the product. • There is no age restriction for using Xailin® Night Lubricating Eye Ointment. • If you use any other eye medicines at the same time, wait for 30 minutes and then apply the Lubricating Eye Ointment. • Use clean tissues to wipe any extra ointment from the eyelids.

Warnings and Precautions: • For external use only. • Replace cap after using. • Do not use with contact lenses. • Do not share Xailin® Night Lubricating Eye Ointment with any other person. • Xailin® Night Lubricating Eye Ointment can cause transient mild burning sensation and/or blurring of vision after application. Wait until your vision is clear before driving or using machinery. • Excessive use of Xailin® Night can cause blurred vision, ocular lubricant infection, irritation and tissue damage. • Do not use after the expiry date shown on the carton or embossed on the crimp of the tube. • Xailin® Night Lubricating Eye Ointment may be used during pregnancy and breast-feeding. • Consult your gynecologist, ophthalmologist or family physician to ensure safety during pregnancy, breast-feeding & use in children. • External ocular use only, do not swallow.

When NOT to use: Do not use if you are allergic to Wool Alcohols or any of the ingredients of Xailin® Night.

Side Effects: • Long term use of Wool Alcohols may cause allergic or hypersensitivity reactions. • Longer contact time may result in mucosal irritation. **Storage:** • Store between 1° and 25°C. Do not freeze. • Discard the contents 1 month after first opening. • Keep out of the reach and sight of children. • Dispose of the used tube according to your local waste management regulations.

Available pack sizes: 3.5 g and 5 g (Not all pack sizes may be marketed).

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FINAL APPROVALS

COUNTRY MANAGER #1:	COUNTRY MANAGER #2:	REGULATORY:
COUNTRY MANAGER #3:	COUNTRY MANAGER #4:	

NOTES: 1) June-2025: (Rev. 2) The VF's address is bad, based on this, DP will not be implemented in the factory and once the NB authorizes the new primary packaging, or if the regular production process requests a change of artwork, we must update all drawings
2) 17-Jun-2025: Since Rev. 2 has not been implemented, and in order to supply the Turkish market, we must update the Partner's address and restore the old VF address

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**EC REP** Crescent Pharma International Ltd.