EN

Description: Carbomer Gel is a viscous and hypotonic solution of 0.2% carbomer 980. Carbomer Gel has excellent mucoadhesive properties and lubricates the eye by evenly coating the corneal and conjunctival surfaces.

Use: For use as lubricating eye drops to alleviate the symptoms of dryness, discomfort, grittiness and minor irritation caused by dry eye.

Each gram contains: Carbomer 980 2,0 mg, Cetrimide 0.1 mg, Sorbitol 40.0 mg, Sodium hydroxide 0.84 mg, Water for injections q.s.

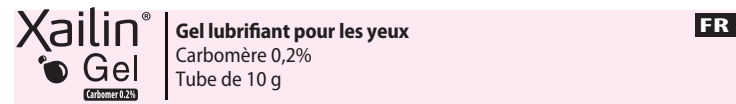
Instructions for Use: Wash hands thoroughly before use. Remove the cap from the tube by lifting the tab on the anti-tamper collar to the cap and peeling the collar off. Tilt the head back and pull the lower lid of the eye to form a pocket. Hold the tube vertically, upside down, and **squeeze gently until a drop forms** at the end of the tube. Place this drop in your eye without touching the eye with the end of the tube. **Blink two to three times to spread the gel evenly over the eye and wipe away any excess gel from around the eyelids.** Close the tube making sure that you do not touch the tip of the tube on any surface (including the fingers).

One drop of the product is instilled into the affected eye(s) 3-4 times daily or as required, depending on the discomfort and relief obtained. The treatment duration can be extended beyond 30 days following consultation with a doctor.

Precautions, warnings and contraindications:

- Do not use if the tube or the security seal is damaged.
- Contact lenses should not be worn during instillation of the product.
- After instillation there should be an interval of at least 30 minutes before reinsertion.
- In case of any additional local ocular administration, there should be an application interval of at least 5 minutes between the use of the two products and Carbomer Gel should be the last product instilled.
- Do not touch the eye or any other surface with the tip of the tube in order to avoid scratching the eye or contaminating the product.
- Replace lid (cap) **gently** after using the tube; **do not overtighten** the cap.
- Do not use the product if allergic to any of the ingredients.
- For external use only.
- A mild transient burning sensation, sticky eyelid, and blurred vision after instillation of the gel have been occasionally reported.
- Keep out of the reach and sight of children.
- Store below 30° C.
- Do not use the product after the expiry date or 30 days after first opening.
- If the problem does not improve within 2 days or worsens at any time, consult your doctor.
- If the problem persists for more than 30 days, consult your doctor.

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Blumont Healthcare Ltd.
MedNet EC-REP IIB GmbH