TREHAPAN®

Colirio estéril sin conservantes ni fosfatos
Preservative and phosphate free sterile eye drops
Konservierungsmittel- und phosphatfreie sterile Augentropfen
Collyre stérile sans conservateur

VISUfarma

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Regenerativo

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EN INSTRUCTION FOR USE

Regenerative

What are TREHAPAN® eye drops? Thanks to the unique combination of three active substances: sodium hyaluronate, trehalose and D-panthenol, TREHAPAN® eye drops provide intense hydration of the eyes and supports the regeneration of the protective barrier on the surface of eye. TREHAPAN® provides long-lasting comfort meaning they need to be used less frequently.

What are TREHAPAN® eye drops used for? TREHAPAN® eye drops are indicated for moderate or severe dry eye, providing intense hydration of the eyes and supporting the regeneration of the protective barrier on the surface of eye. They accelerate the regeneration of the surface structures of the eye after surgery.

When should I use TREHAPAN® eye drops? TREHAPAN® drops should be used in conditions of discomfort, accompanied by symptoms such as: eye dryness, burning or tired eyes, which may be caused by: • dry air in heated or air-conditioned rooms, prolonged screen use, watching television for long periods or cigarette smoke; • mechanical irritation of the eye surface, e.g. from wearing contact lenses; • surgical procedures (including laser ones) that interfere with the processes of secretion and proper distribution of the tear film on the surface of the eye.

How do the active ingredients of TREHAPAN® eye drops work?

• **Sodium hyaluronate:** a naturally occurring polysaccharide in the human body, including the vitreous body of the eye, is commonly known as a highly moisturising substance. The mucoadhesive properties of hyaluronic acid ensure better adhesion of the preparation to the eye surface, which contributes to faster healing of the corneal epithelium, while thanks to the viscoelastic properties, it increases the stability of the tear film, which is washed out much slower during blinking.

• **Trehalose:** is a non-reducing disaccharide composed of glucose units, naturally occurring in many living organisms such as plants, insects, fungi and bacteria. It has been identified as a key component of survival under stress related to anhydrobiosis (almost complete dehydration). Due to its strong protective properties, trehalose has been used in ophthalmological preparations as a substance preventing the stress associated with dehydration of the eye surface, which is a consequence of, among others, dry eye syndrome.

• **D-panthenol:** is an alcohol analog of pantothenic acid, also called pantothenate or vitamin B5, which is a water soluble vitamin. Due to its hygroscopic properties and the ability to retain moisture, the topical application of D-panthenol may improve hydration, the effect of which is related to the ability to regenerate the protective barrier.

Who are TREHAPAN® eye drops intended for? TREHAPAN® eye drops are intended for adults.

How often should TREHAPAN® eye drops be used? TREHAPAN® eye drops can be used daily, including with all types of contact lenses, as the drops do not contain any preservatives. It is recommended to apply one drop into the conjunctival sac of the eye 3-4 times a day. You can apply more often if needed.

How to use TREHAPAN® eye drops? TREHAPAN® eye drops are for external use only to be applied to the conjunctival sac of the eye.

When TREHAPAN® eye drops are contraindicated?

• Do not use in the case of hypersensitivity to any of its components.

• Do not use if you are allergic to the ingredients.

What do TREHAPAN® eye drops contain? Sodium hyaluronate 0.15%, trehalose 3%, D-panthenol 2%, sodium chloride, highly purified water.

How to store TREHAPAN® eye drops?

• Keep out of sight and reach of children.

• Keep away from sources of heat.

• Store in a dry place at a temperature of 5°C to 35°C.

• Do not use the product after the expiration date printed on the package.

• The shelf life of the product is 3 months after first opening the container.

• 1 month treatment following the recommended dosing. Discard the bottle 3 months after first opening.

• Discard any unused solution.

Warnings and precautions:

• The preparation is intended for external use only.

• Do not use if package is damaged.

• Never touch your eye or any other surface with the tip of the dropper, this may contaminate the solution.

• In the case of using other ophthalmic solutions should wait at least 15 minutes before the application of TREHAPAN® eye drops.

• The preparation should be administered last.

• The reusable medical device should be used by one patient.

Reporting medical incidents or adverse reactions: If you get any side effects, please tell your doctor, pharmacist or nurse.

Package: Bottle of 10ml in carton unit box.

Date of instruction preparation: June 2023

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8-9-10-11) After use, place the protective cap on the dropper. During application, be careful not to touch the surface of the eye and eyelid with the tip of the dropper. After applying the drops, do not rinse the surface of the eye with water or any other liquid. Remember that a few seconds may pass between the squeezing of the bottle and the flow of the drop in. Do not apply too much pressure. Before closing, point the bottle with the applicator downwards and shake vigorously, without touching the tip of the dropper, in order to remove the remaining drops. Wait a moment before putting the cap on for the bottle to vent.

SUPROBION Sp. z o.o., Sp. k.
ul. Kiełcowska 2 55-095 Mirków,
Polonia/Poland/Polen/Pologne

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FR 1) Prior to administering the preparation, wash hands thoroughly. **2)** Remove the cap and apply into the conjunctival sac of the eye. Do not touch the surface of the eye with dropper tip. **3)** Tilt your head back slightly. **4)** Hold the bottle face down (vertically). **5)** Pull your lower eyelid down slightly, pressing the bottle, squeeze one drop into the conjunctival sac. **6)** Close your eyelids and spread a drop across the eye surface by blinking. **7)** Repeat the same activities for the other eye. **8)** Put the protective cap on the dropper after use. During applying, be careful not to touch the surface of the eye and eyelid with the tip of the dropper. **9)** After applying the drops, do not rinse the surface of the eye with water or any other liquid. **10)** Remember that a few seconds may pass between the squeezing of the bottle and the flow of the drop in. Do not apply too much pressure. **11)** Before closing, point the bottle with the applicator downwards and shake vigorously, without touching the tip of the dropper, in order to remove the remaining drops. Wait a moment before putting the cap on for the bottle to vent.

EN 1) Prior to administering the preparation wash hands thoroughly. **2)** Remove the cap and apply into the conjunctival sac of the eye. Do not touch the surface of the eye with dropper tip. **3)** Tilt your head back slightly. **4)** Hold the bottle face down (vertically). **5)** Pull your lower eyelid down slightly, pressing the bottle, squeeze one drop into the conjunctival sac. **6)** Close your eyelids and spread a drop across the eye surface by blinking. **7)** Repeat the same activities for the other eye. **8)** Put the protective cap on the dropper after use. During applying, be careful not to touch the surface of the eye and eyelid with the tip of the dropper. **9)** After applying the drops, do not rinse the surface of the eye with water or any other liquid. **10)** Remember that a few seconds may pass between the squeezing of the bottle and the flow of the drop in. Do not apply too much pressure. **11)** Before closing, point the bottle with the applicator downwards and shake vigorously, without touching the tip of the dropper, in order to remove the remaining drops. Wait a moment before putting the cap on for the bottle to vent.

Distribuido por/ Distributed by/ Vertrieb durch/Distribué par:
VISUFARMA B.V., Amstelplein 1, 1096 HA, Amsterdam,
Paises Bajos/ Netherlands/ Niederlande/ Pays-Bas
info@visufarma.com

MD STERILE A

AW-00636 Rev. 1/11389A_NL